

UltiCare[®] VetRx

iPet[™] PRO



ANIMAL HEALTH DIABETES CARE

Optimizing care for
pets with diabetes



Blood Glucose Monitoring System

Fast results,
proven accuracy

Advancing technology for improved accuracy

Accurate glucose data is essential for those who treat and manage pets with diabetes. The iPet PRO is designed to analyze both venous and capillary blood samples, providing better continuity of care between pet owners and veterinarians.


- 98% system accuracy with a small blood sample
- Results in just 5 seconds
- Analyzes venous and capillary blood samples



REMARKABLE ACCURACY, TIME AFTER TIME.

The iPet PRO blood glucose meter provides results within ± 15 mg/dL 98% of the time, exceeding the minimum acceptable accuracy criteria specified in ISO 15197:2013.

Consensus Error Grid (CEG)

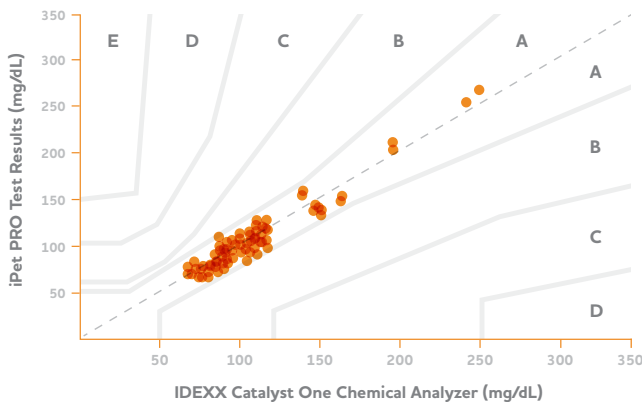

CONSENSUS ERROR GRID (CEG) %						
TOTAL	REGION	A	B	C	D	E
94	Points	94	0	0	0	0
	CEG %	100%	0.0%	0.0%	0.0%	0.0%

The consensus error grid shows that 94/94 (100%) values fall within Zone A, defined as "no effect on clinical action"¹

¹ International Organization for Standardization. In vitro diagnostic test system requirements for blood-glucose monitoring systems for self-testing in managing diabetes mellitus. 2013. ISO 15197:2013.

Product	UPC#	Product	UPC#
iPet PRO Blood Glucose Monitoring Kit*	3-57515-52000-5	iPet PRO Test Strips, 50CT	3-57515-52002-9
iPet PRO Test Strips, 25CT	3-57515-52001-2	iPet PRO 28 Gauge Lancets, 100CT	3-57515-52004-3

*Kit includes blood glucose meter, 25 test strips, glucode chip, 30 lancets, lancing device, control solution, 2-AAA batteries, carrying case, user guide, user quick guide and logbook.

UltiGuard® Safe Pack

Dispense + Dispose

The only FDA cleared dual-purpose system that includes 100 premium quality syringes and a sharps container for a single cost.

How UltiGuard Safe Pack Works


The patented dual-purpose system allows you to dispense a new syringe and safely dispose used syringes. It's the only all-in-one solution to keep healthcare providers, pet owners, pets and communities safe.

The Safe Pack meets proper disposal guidelines in all 50 states. For a list of guidelines, visit ulticare.com/state-disposal-guidelines.

DROP
the used syringe
in the top

ROLL
the handle to discard
the syringe

POP
out a new syringe

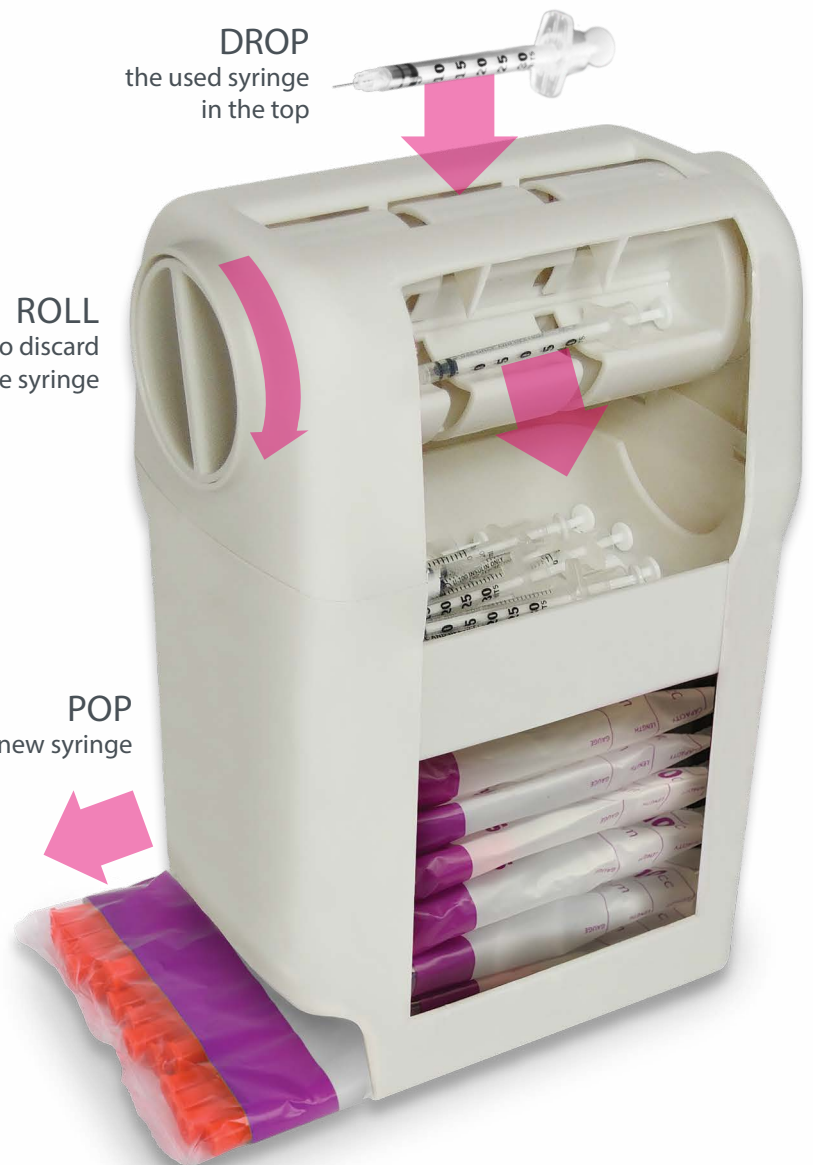

The UltiGuard Safe Pack is available with syringes or pen needles.

UltiCare VetRx Insulin Syringes

Available in the UltiGuard Safe Pack—the only dual-purpose system that allows you to dispense a new syringe and safely dispose used syringes. The patented all-in-one solution is designed for clinic or in-home use.

The Safe Pack contains 100 syringes and is available in 3/10cc, 1/2cc and 1cc sizes in a variety of needle gauge and lengths. Also available in 60 and 100 count boxes.



U-40

3/10cc (12-UNIT)

ULTIGUARD 100-COUNT	BOX 100-COUNT
29G x 12.7mm (1/2")	
UPC# 3-57515-07263-4	UPC# 3-57515-09263-2
29G x 12.7mm (1/2") with half-unit marking	
UPC# 3-57515-82003-7	UPC# 3-57515-82001-3
_____	_____

1/2cc (20-UNIT)

ULTIGUARD 100-COUNT	BOX 100-COUNT
29G x 12.7mm (1/2")	
UPC# 3-57515-07260-3	UPC# 3-57515-09260-1
29G x 12.7mm (1/2") with half-unit marking	
UPC# 3-57515-82004-4	UPC# 3-57515-82002-0
_____	_____

1cc (40-UNIT)

ULTIGUARD 100-COUNT	BOX 100-COUNT
29G x 12.7mm (1/2")	
UPC# 3-57515-07261-0	UPC# 3-57515-09261-8
_____	_____

U-100

3/10cc (30-UNIT)

ULTIGUARD 100-COUNT	BOX 100-COUNT	BOX 60-COUNT
31G x 8mm (5/16")		
UPC# 3-57515-92001-0	_____	_____
31G x 8mm (5/16") with half-unit marking		
UPC# 3-57515-07436-2	_____	UPC# 3-57515-09436-0
29G x 12.7mm (1/2")		
UPC# 3-57515-07231-3	UPC# 3-57515-09230-4	_____

1/2cc (50-UNIT)

ULTIGUARD 100-COUNT	BOX 100-COUNT	BOX 60-COUNT
31G x 8mm (5/16")		
UPC# 3-57515-92002-7	_____	_____
29G x 12.7mm (1/2")		
UPC# 3-57515-07251-1	UPC# 3-57515-09250-2	_____
28G x 12.7mm (1/2")		
_____	UPC# 3-57515-08250-3	_____

1cc (100-UNIT)

ULTIGUARD 100-COUNT	BOX 100-COUNT	BOX 60-COUNT
29G x 12.7mm (1/2")		
_____	UPC# 3-57515-09210-6	_____
28G x 12.7mm (1/2")		
_____	UPC# 3-57515-08210-7	_____

U-40 insulin syringes are used with Vetsulin[®], ProZinc[®] or any U-40 strength of insulin.


U-100 insulin syringes are used with Levemir[®], Lantus[®], Humulin N[®] or any U-100 strength of insulin.



Half-Unit Marking Insulin Syringes

Half-unit marking syringes are designed for precise and accurate dosing—making it easier to administer small or half-unit doses.

Available for U-40 and U-100 insulin in a variety of needle gauge and lengths.

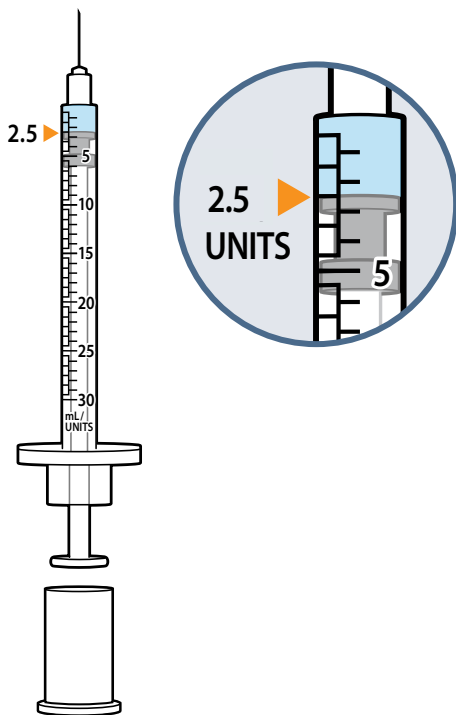

Shown as
U-100, 3/10cc
31G x 8mm (5/16")

UltiCare VetRx Pen Needles

Available in the UltiGuard Safe Pack—the only dual-purpose system that allows you to dispense a new needle and safely dispose a used needle.

The Safe Pack contains 100 premium quality needles and is available in the 12.7mm size. UltiCare VetRx pen needles fit all pen injectors sold in the U.S.

ULTIGUARD
100-COUNT

12.7mm (1/2") x 29G

UPC# 3-57515-72002-3

For use with Vetsulin VetPen®, Levemir® FlexTouch®, Lantus® SoloStar® and Humulin N® KwikPen® insulin pen injectors.

UltiMed Incorporated

is a privately held company that manufactures premium quality insulin syringes, pen needles, and safety syringes for the diabetes and animal health markets. The company was founded in 1988 to create high quality products at a better price.

The UltiGuard Safe Pack by UltiMed, is the only FDA cleared dual-purpose system that combines premium quality needles and a sharps container. UltiMed's exclusive all-in-one solution makes the company the world leader in safe needle disposal.

UltiMed is based in Minneapolis, Minnesota with manufacturing facilities in the United States and abroad.



ULTIMED
INCORPORATED

ulticare.com

1.877.854.3434